

WASH Instructions

1

Please read and adhere to the following guidelines provided before using iQprint® Wash

1. Ensure your wash machine is off
2. All previous resin cleaner has been removed if using iQprint® Wash for the first time
3. Take care not to spill the cleaner onto any parts or components
4. Wear all required PPE, including a lab coat or overall, long gloves (not biodegradable) and appropriate safety glasses
5. The wash unit or bowl must be on a flat, stable surface
6. The wash unit or bowl must be completely clean and free from contaminants
7. Fill the wash unit or bowl to the appropriate level
8. iQprint® Wash should not be hot when used
9. Submerge or clean your models for a minimum of 3 and a maximum of 10 minutes
10. If required, remove the excess materials with a soft bristle brush
11. Rinse with warm water
12. Allow to dry for 1 hour or alternatively use compressed air or a drying cycle in your curing unit
13. The 3D printed parts must be dry to touch
14. Once dry, please follow the curing instructions of the resin manufacturer
15. Wash your hands immediately after use if they come into contact with the wash detergent